

Recording Messages into the Dec-Aid

- Plug the switch into the socket location you want to record a message into.
- Turn VOL knob to ON, but keep it at a very low volume (near OFF position)
- Make sure the switch on the side is in the REC (for 'record') position (not PLAY)
- Touch switch (and keep it held down) and speak message towards top of Dec-Aid with your mouth around 6" from the device.
- It will repeat your message back to you.
- If you don't like it, record the message again. (You can vary the distance of your mouth from the Dec-Aid, the loudness of your voice, and the position of the volume knob until the sound is OK.)
- Once you are happy with the message, switch back from REC to PLAY. Use switch to play the message.
- Repeat for each of the 10 sockets, as required.
- It is useful to write down the stored messages on a 1-10 numbered record sheet, stored alongside the device, to save having to try them all out each time to remind yourself what is in there!

If the Dec-Aid stops working, try replacing the batteries (4 x the slim AAA batteries). You need to use a Philips screwdriver to open the battery compartment.

