

Basic Device Controls

Turning on

At the top right hand corner of the Dynamo (on the blue surround) there is a button, press down firmly on it with your thumb. A green light will briefly come on and the device will brighten. The Talker screen will open at the page it was shut down at.

Shutting down the Dynamo

There are 2 ways to shut down the Dynamo

Press the power button (the one you used to turn the device on). A message will appear on screen saying "*Are you sure you want to shutdown?*" Press *Yes*.

Press the *set up* button (top right hand button on the touch screen/ control panel) the Dynamo Setup menu will appear. On the bottom right hand corner there is a Shut Down button, press the message "*Are you sure you want to Shutdown?*" press *Yes*.

Button Modifying Options

Recording a message into a button

Press the *Record* button (top left hand corner of control panel with microphone)

Press the button on the screen that you want to record a message into and then lift your finger off it.

Speak clearly, and your message will be recorded (and all background noise!)

Press the button again to stop the recording.



Modifying/ changing a symbol

Press the *modify* button (top left hand corner next to record) on the control panel - nothing happens - but carry on -

Immediately press the symbol button that you want to change.

A *Modify Button* screen will appear. Press the *Modify Symbol* button, in the second column from the left.

On the top left hand corner there is a *Search* button, Press that, then type in name of the new symbol you want to put in, instead & press *OK*. A list will appear, press the symbol you prefer.

You will go back to the *Modify Button* screen and the new symbol will be next to the modify symbol button. Press *OK*.

Modify the label/ writing on a button (keeping the same symbol)

Press the *modify* button.

Press the button you want to change.

Press the *modify label* button (middle of second column from the left)

An *edit label* screen will appear; press *clear* to delete the current label. There are *cap locks* in the bottom left hand corner (they return to lower case after one letter). Press *OK* when you have finished.

You will go back to the previous screen. Press *OK* to close the *modify button* menu.

Swapping the location of a button

Press the *modify* button in the control panel.

Press the button you want to move.

Press the *swap location with another button* (middle of far left hand column)

Press the other button what you want to move, they will automatically switch places.

Changing the font size of a button

Press the *modify* button.

Press the button on which you wish to change the font size.

Keep pressing the *font size* button until you find the size you want e.g. *small, medium*.

Press OK to close the *modify button* menu.

Modify the page link of a button

Press the *modify* button

Press the button you want to modify.

Press *modify page link* (next to *cancel* button). This will take you into *Select Page*.

You can set the page link by

using the scroll up/ down and the arrow up/ down buttons to find the page name you want

pressing the *set link to previous page*.

or to delete the page link press *cancel*.

5. Press OK to close *modify button* menu.

Making a new page and naming it

Press the *Set up* button (top right hand corner)

On the Dynamo setup Page press the *make new page* Button (middle of top row)

You then choose how many rows and columns you want by touching the *more* and *fewer* buttons. Look for the *row and column* at the top this will change when you touch the *more/ fewer* button. Also you can change the font size by using the *Font Size* button.

Press *OK*. Now you will need to give the page a name, just type it in, pressing *OK* at the bottom left hand corner when you have finished.

Now you can record/store your messages into the page.

Linking your new page

Your new page will not be linked to any other page. You need to decide where you want the link to be. Press *Modify*.

Press *modify page link*.

You can then use the arrows or the scroll buttons to find the page you want to link with, or just set to previous page.

Press *OK* to go back to *Modify Button*

Press *OK* to go back to the page you created. There will be a button that looks like a folder - that's the link.

Press the link to go back to where you set the link from.