



This is a 4 cell direct/switch operated scanning board which offers visual and auditory prompts to enable users to choose from up to 16 different spoken messages (4 levels) or operate up to 4 battery operated toys/devices. Can be set to use single or two-switch scanning as well as direct mode.

### Record a message

Switch POWER ON. Set SCAN to OFF and AUDITORY to OFF.

Hold coloured pad and record button. You will hear the prompt 'record on' to reassure you that you are ready to record. Keep holding finger down on pad and speak (microphone is at top left of front). When you are finished take finger off and you will hear 'record off'.

### Record an auditory prompt

Switch POWER ON. Set SCAN to ON and AUDITORY to ON.

Scanning will start and red light move across the board. When light appears above pad then press pad and record message - needs to be short though as you only have 2 seconds!

### Scanning modes

**Auto-scan:** moves automatically across the board, light goes on as each is active, user presses switch to select and the message or action will start. Uses GREEN switch socket.

**Step-scan:** Uses YELLOW socket to start scan and GREEN to select.

**Dwell-scan:** Uses YELLOW socket to quickly move across and hold to activate.

**Scan speed and volume** can be increased or decreased by holding down button at back.

### Attach a toy

Plug into coloured socket on right side of back. 'Adapting a battery toy' quick guide can be found on the CALL website in the Resources section.

